



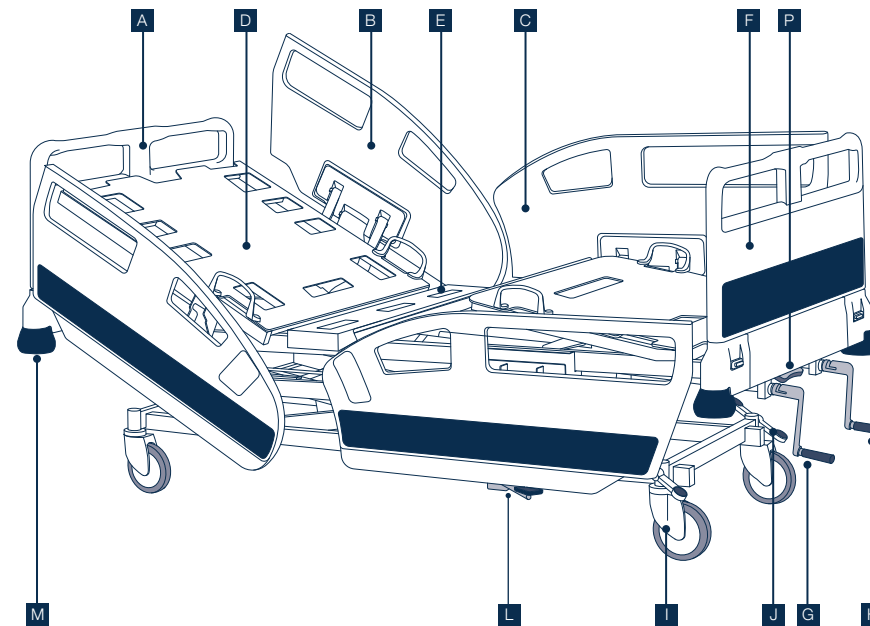

NewCare V1 Bed

QUICK REFERENCE SHEET

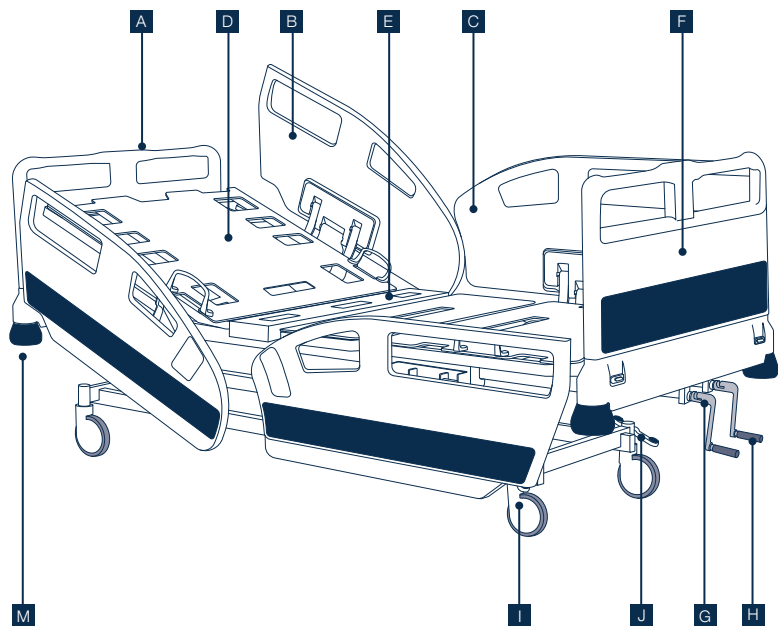

BED'S COMPONENTS

A	Removable headboard	H	Backrest spindle
B	Backrest side rails	I	Integral wheels (Ø150 mm.)
C	Legrest side rails	J	Braking pedal.
D	HPL Mattress platform	L	Lifting pedal
E	Sheet steel central section	M	Protection discs
F	Removable footboard	P	Trend / Reverse trend handle
G	Legrest spindle		

CLEANING

Daily cleaning and disinfection with soapy water:

- Rails. Head board. Foot board.

Cleaning and disinfection after a patient leaves or before a transfer:

- Mattress platform area. Chassis of the bed.

Thorough cleaning and disinfection every two months:

- Support and lifting area of the bed.

MAINTENANCE

Always check the correct state of:

- Mechanical drives.
- Wheels.
- Braking mechanisms.

www.pardo.es

TECHNICAL SERVICE CENTRE

(+34) 976 300 033

For more information, please consult the use and maintenance manual.



MECHANICAL FUNCTIONS


SECTIONS MOVEMENTS

Unlock crank:


- Pull the lever bed outwards.
- Direct it to its working position.
- Push the lever to the correct position.

Articulating sections:


- Raise: Turn clockwise the left crank handle..
- Lower: Turn anticlockwise the left crank handle.

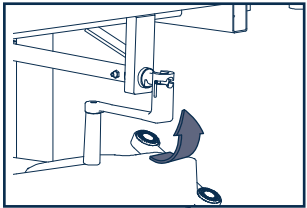


Unlock crank

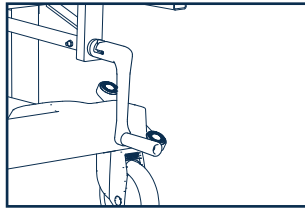

Working position

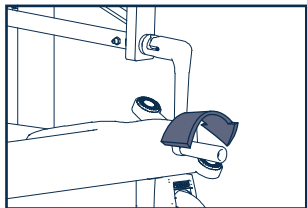

Rise the sections

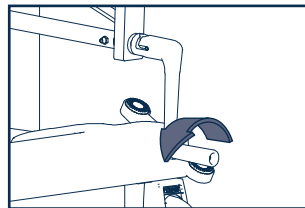


Lower the sections

TRENDELEBURG / REVERSE TRENDELEBURG POSITIONS

Trendelenburg tilting:

- Press the handle placed on footboard.
- Tilt the footboard related to the headboard until the maximum inclination is reached.

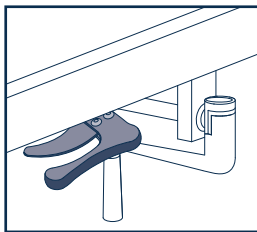


Reverse trendelenburg tilting:

- Press the handle placed on footboard.
- Tilt the headboard related to the until the footboard maximum inclination is reached.


OTHER FUNCTIONS

Side Rails (1):


- Raising the rail: Pull the rail upwards until it is immobilised in the highest position.
- Lowering the rail: Pull the handle found underneath the rail outwards. The rail will automatically lower.

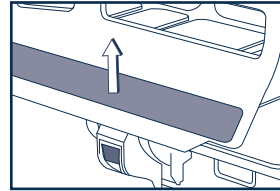

Raising side rails

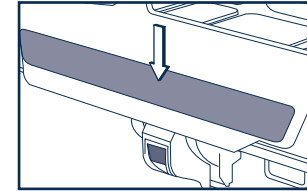


Lowering side rails

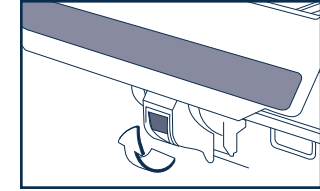


Centralised braking system (2):


- Braking the bed: Pedal in low position.
- Moving the bed: Pedal in intermediate position.
- Activating steering wheel: Pedal in raised position.

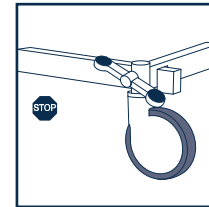

Brake

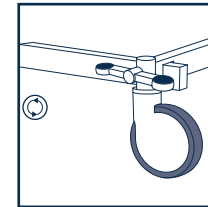

Free movement

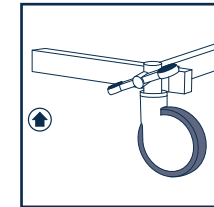


Steering wheel

HEIGHT ADJUSTMENT


Raise mattress platform:

- To operate the rising movement is stepping several times the green pedal until the bed arrives to the desirable height.

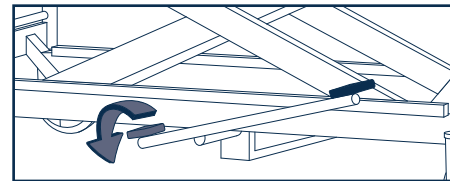

Lower mattress platform:

- To operate the lowering movement is stepping red pedal.
- If the red pedal is kept depressed, the bed lowers to its lowest position.

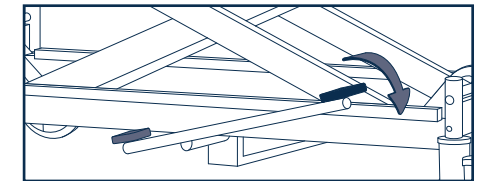

SAFETY WARNINGS

- The maximum duration of a continuous manoeuvre should not exceed 2 minutes, as damage may be caused to the equipment.
- Before raising and lowering the moving parts of the mattress platform, with the other parts of the bed or accessories, it is essential to check that there are no obstacles between them.
- The Trendelenburg and reverse Trendelenburg movement should always be performed under medical supervision.

