

SWL: 250 kg/551 lbs

TurningSling is an assistive device for turning patients that is used in combination with a two-point slingbar mounted on an overhead or mobile lift. It is designed to provide support at the lower spine while allowing the seat to remain open, thereby facilitating hygiene and dressing. TurningSling has two suspension points, one on each side. TurningSling is made of a durable disposable fabric, which means that it cannot be laundered and must be discarded when it becomes soiled or is no longer required by the patient.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The slings are available in several different materials and in sizes from XXS to XXL. All models are safe and very easy to use and are rated for lifting patients weighing up to 300 kg. The choice of model and material depends on the patient's needs and the transfer situation.



Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.



Always read the manual

Always read the manuals for all assistive devices used during a transfer.


Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.

The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your dealer if you have any questions.

Description

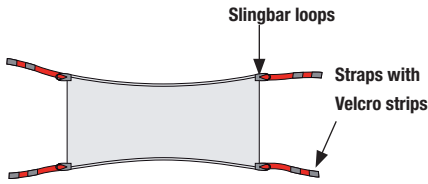

TurningSling is made of a durable disposable fabric and has two slingbar loops and two straps with Velcro strips on each end.




Note! Straps with Velcro strips must not be fastened to the slingbar!

Note! The product may only be used for turning patients.


Note! The bed side rail must be raised on the side towards which the patient is to be turned.

Placement

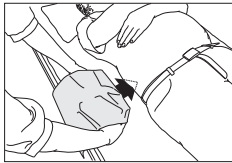

Fold in one end of the TurningSling and place your hands in the pocket that is formed.

Place the TurningSling under the patient's lower spine.

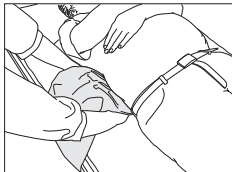

Press the fabric down towards the mattress and push it in under the patient as far as possible.




Go to the opposite side, place your hands under the patient, and then grasp the TurningSling and pull it out.



Pull until there is an equal amount of fabric on both sides of the patient.

Placement with EasyGlide Oval





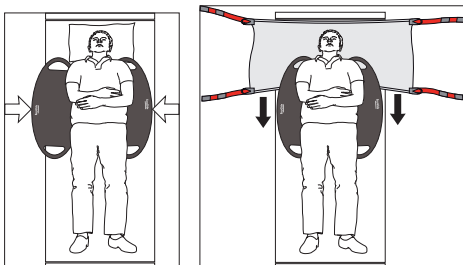


Place two EasyGlide Ovals under the patient.

Place the TurningSling between the sliding boards and the underlying surface. Then, pull the TurningSling down behind the patient's back.

Remove the sliding boards.

Use

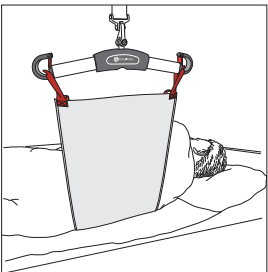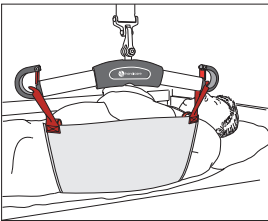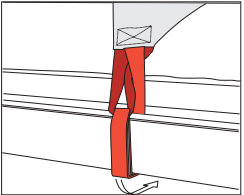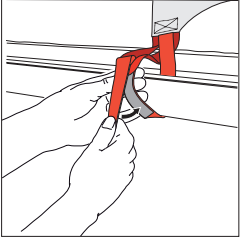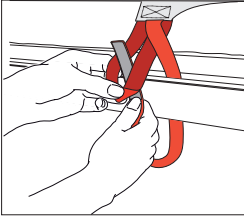

If the TurningSling is to be left under the patient after turning, one side of it must be secured to an appropriate part of the bed frame.

The loose strap with Velcro is secured to one side of the bed frame and the loop to the other side.

Guide the strap with Velcro through the loop and fasten the Velcro.

Check to ensure that it is securely fastened by applying tension to the strap.

Prepare the patient for turning.

Raise the bed side rail on the side towards which the patient is to be turned. Hook the loops to the slingbar. Begin turning, but pause and check to ensure that everything is as it should be. Continue turning until the patient is lying on one side.

Note! Straps with Velcro strips must not be fastened to the slingbar!

Allow the lift to hold the TurningSling in position to provide back support for the patient during the hygiene/dressing procedure.

Note! The product may only be used for turning patients.

Application of ThoraxSling on MiniLift

Pull one end of the TurningSling, so that it slides under the patient.

The TurningSling can also be left under the patient if it is to be used often. It can also be bedded under a draw sheet.

Material:

Non-Woven: Polypropylene, hydrophobic

Care of the product, Non-woven:

Read the product label.

Do not launder. Discard the product if it is soiled or if there is a need to use another product.



Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Handicare AB

Veddestav. 15. Box 640

SE-175 27 Järfälla

SWEDEN

Tel: +46 (0)8 -557 62 200

Fax: +46 (0)8-557 62 299

Internet: www.handicare.com

E-mail: info@handicare.se

