

# DoubleX K™

Complete FluidiXX Management System  
For SERIES 20000® Legacy®

## Directions for Use

1. Hold the cassette by the handle and angle it toward you (over the lip on bottom of the housing on the side of the machine). Press down and insert the cassette into the housing, using a single motion. Gently pull apart the sides of the drain bag to allow it to hang freely.
2. Spike the irrigation bottle. Squeeze the drip chamber to fill it approximately half full.
3. Connect the female luer on the aspiration line to the male luer on the irrigation line. The aspiration line is designated with a red stripe on the Hi-Vac tubing, and a blue stripe on the standard tubing.
4. Select Irrigation Footswitch mode. Press the TEST button. Then press the PRIME key on the front panel or the TEST key again to initiate the priming sequence. If the cassette fails the prime test, please refer to the manufacturer's Operator's Manual.

Please refer to the manufacturer's Operator's Manual for further instructions on the priming and tuning cycle of the Legacy Phaco Machine.

**IMPORTANT:** The XXK Cassette Directions for Use are to be read and used in conjunction with the manufacturer's Operator's Manual. Failure to read the manufacturer's Operator's Manual could result in the improper use of this product.

Please notify Medicel immediately if one of the components is received defective.

Manufactured by:

**Medicel AG**

Dornierstrasse 11

9423 Altenrhein

SWITZERLAND

Tel.: +41 71 727 10 50

Fax: +41 71 727 10 55

E-mail: [info@medicel.com](mailto:info@medicel.com)

Website: [www.medicel.com](http://www.medicel.com)

**CE 0482**

Series 20000® and Legacy® are trademarks of Alcon Laboratories, Inc.